Meine Rezepte
my recipes

Jan Schu

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With NOVACer® I have a complete system of materials in my hands which have been tested and created by using it over and over in daily practise. This is why we call it "MADE BY EXPERIENCE".

All materials complement each other.

NOVAZir® is the brand name for all the different zircon options that are available and NOVACer® ZR is the matching layering ceramics.

NOVAPress® which is available in a wide range of press pallets can be veneered with NOVACer® LDS ceramics.

Completing the range is the classical metal ceramics NOVACer® MC.

My preference for the refractory die technique is also.

Constantly predictable outcomes are very important to me and this total system allowed me to produce beautiful natural smiles for my patients.

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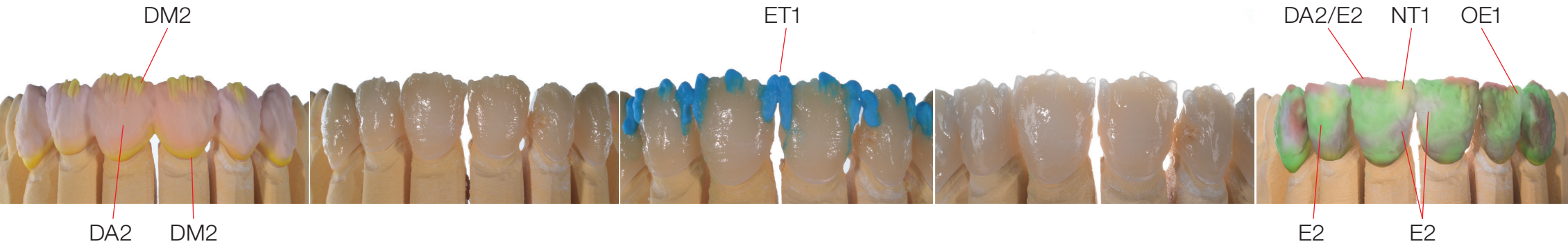
*For single crowns, a translucent zirconia such as NOVAZir® Max.T is particularly suitable.
In the cervical area, a chroma dentin such as DM2 is applied. The tooth form is built up with dentine.*

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
On top of the dentine cutback a thin layer of internal mamelons are placed. After the first bake ET1 is built up and fired, that way it is easily controlled. The restoration is now completed with different enamel and translucent powders.

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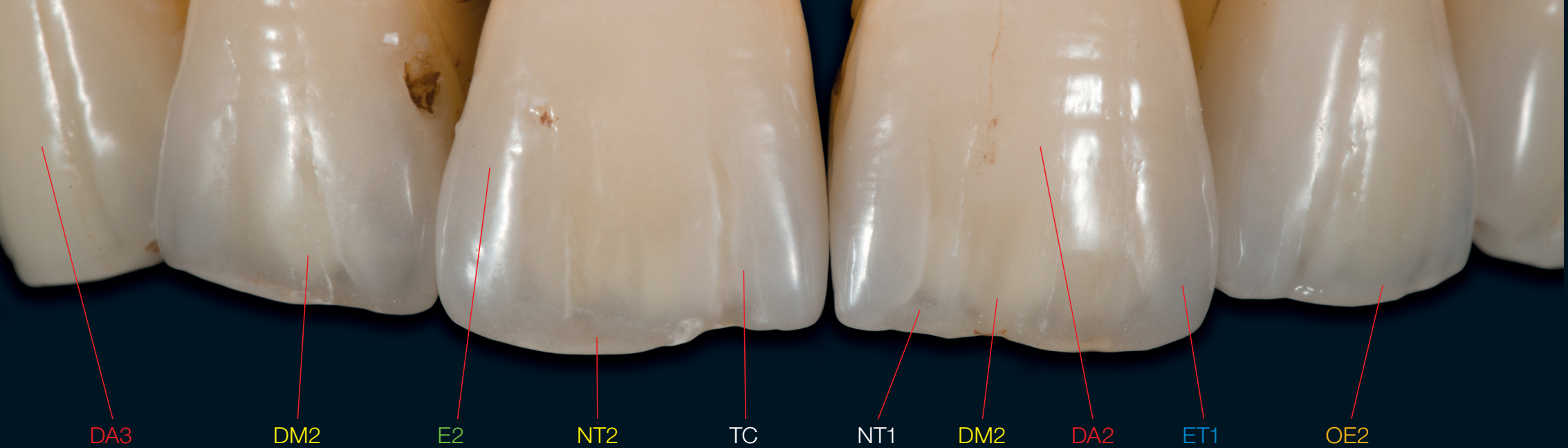Texture and surface lustre is achieved using diamonds and rubber wheels.

DA3

DM2

E2

NT2

TC

NT1

DM2

DA2

ET1

OE2